

# Designed Specifically for Mounting



Introducing

**Denar®**

**LabRelator**

For Mark 300 Series  
Articulators

- Helps extend the life of your current Mark 300 series instrument
- Fully interchangeable with Mark 300 series articulators, which have factory set accuracy to within 20 microns
- Upper member is non-removable and does not allow excursive movements
- Durable anodized finish for easy cleaning
- Smaller size for easy handling



Mark 310



Mark 320



Mark 330

Call **800-626-5651** today  
to find out how easy the  
LabRelator is to use.  
[www.whipmix.com](http://www.whipmix.com)



RESTORATIVE ORAL HEALTH

# Made to last. Mount as often as you like.



## LabRelator Instructions

- Seat the maxillary cast in the facebow registration and make sure it is stable with no rocking.
- Open or invert the LabRelator so that you can relate the lower cast to the upper cast. The thumbscrews in the back of the LabRelator are for stabilization when inverted.
- If you are using an interocclusal record, seat it in place between the two casts.
- Stabilize and mount the mandibular casts. FlexiMount Universal Mounting Support is ideal for this purpose.
- Transfer the casts with confidence to any Mark 300 series articulator for diagnosis and treatment planning.

**Item 20011380 LabRelator with 10 pack of magnetic mounting plates**

**For more information,  
or to view our Mark 300  
series video, visit us at  
[www.whipmix.com](http://www.whipmix.com).**



RESTORATIVE ORAL HEALTH

Whip Mix Corporation

361 Farmington Avenue, Louisville, KY USA 40209

800-626-5651 • 502-637-1451

Fax: 502-634-4512

[www.whipmix.com](http://www.whipmix.com)

