ADVANCED LABORATORY SYSTEMS

primopattern LC Gel / Paste

lichthärtendes Universal- Modellierkomposit
light cured universal modeling resin

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product description

primopattern is a light cured, ready to use one-component-material that is available either as modelling gel or modelling paste. As a universal composite modelling resin, primopattern was developed for a large variety of uses in dental technology.

It works excellent in the different areas of implantology, all ceramics (copy milling, CAD/CAM), bite transfer and bite control, fixed and removable restorations as well as all sorts of fixings and blockings.

As modelling gel, primopattern is applied directly from the syringe, as modelling paste the material offers a clay like consistency and can easily be formed, applied or adjusted. primopattern cures in most conventional light curing units and burns out cleanly and completely even if thick layers had been built up.

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product characteristics and advantages

primopattern Gel and Paste stand out due to the following product characteristics:

- ready to use one-component-material, light cured and thixotrop
- perfectly aligned viscosity of Gel and Paste for quick and precise application
- stays where placed – does not run – therefore very economical
- easy control of the applied thickness
- long working time (> 20 min.) – short curing time (< 5 min.)
- cures in most conventional light curing units (320nm-500nm)
- polymerisation neutral and absolutely dimensionally stable with great fitting properties
- no distortion, high surface hardness and great strength
- light cured cases can be lifted off easily and stress free
- easy to grind with carbide burs or rubber polishers
- burns out cleanly and completely
- can be used in conjunction with conventional waxes or light cured wax (metacon)
- does not contain MMA and peroxide and has no taste or smell

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application areas

• Implantology:

- modellations for casting or scanning of
 - implant abutments
 - implant bars
 - implant superstructures
 - tertiary constructions over Galvano mesiostructures

production of

- transfer- or insertion guides
- implant jigs, verification indexes, etc.

as well as connecting or fixing of single implant parts or segments

• fixed restorations:

- modellations for casting, scanning or pressing of
 - crowns and bridges
 - understructures for zirconium copy milling machines
 - Inlays and Onlays
 - understructures for Galvano or Captek bridges

• combined restorations

- primary and secondary telescopic or conus crowns
- individual attachments
- RSS secondary attachments, etc.

• bite transfer and bite check (control)

- transfer splints
- bite check splint
- stops and checkbites

• further uses

- retaining single parts or parts that needed to be separated for welding or soldering
- fixing of attachments
- general retaining or fixing of single parts or segments

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polyurethane model and die material: use a thin layer of Vaseline

polished metal or zirconium surfaces normally don't require any separator. In critical cases apply a finest layer of Vaseline or use oil and blow off any excess.

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layer thickness

In case you would like to build up a thick layer of primopattern LC Gel, don't build up the desired thickness in one time but rather build up several thinner layers (approx. 1 mm per layer) on top of each other. Pre cure each layer either for about 10 seconds with a handheld light or one minute in the light curing unit, before applying the next layer. This way you avoid that due to one too thick layer incomplete polymerisation, increased heat release or curing tension might occur. **This working procedure is not needed for primopattern LC Paste.**

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oxygen inhibition layer

the easiest way to remove the inhibition layer (surface stickiness) is to wipe it off with primoclean cleaning liquid (primotec, item number PS205). Alternatively use an alcohol based cleanser.

Attention: in order to achieve best casting results and surface qualities the oxygen inhibition layer needs to be wiped off completely prior to investing.

When working with several primopattern

LC Gel layers (as described above), the oxygen inhibition layer between the primopattern layers must not be removed! The oxygen inhibition layer grants a perfect bond between the material layers and avoids material tension.

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metalights mini, trend & classic

suitable light curing units and polymerisation times

- primopattern cures in most conventional light curing units that supply a wavelength spectrum of 320nm-500nm.

Most suitable are the Metalight light curing units (primotec), as they are equipped with a special cooling system for most gentle polymerisation.

average polymerisation times in light curing units equipped with:

- UV/UV-A lights (i.e. metalight): 3,5 – 5 minutes
- halogen lights: 2 minutes
- strobo lights (i.e. UniXS): 90 seconds

polymerisation time was measured on samples with > 4,5mm layer thickness. Please note that the light curing times may vary due to quality and age of the light bulbs in the unit. Constant curing times can only be secured when the light curing unit is maintained properly and the bulbs are renewed on a regular basis. Please check the maintenance suggestions of the particular curing light manufacturer.

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basic material handling

- The screw cap of the primopattern LC Gel syringe is removed prior to use and one of the included hollow needles is attached instead. With this hollow needle primopattern LC Gel can be applied easily and right to the spot. After use the syringe is sealed again with the screw cap.

For modelling hold the syringe as shown in the pictures and lightly press the piston. When the syringe is used with a lightly vibrating motion, the material becomes more flowable, when the vibrating motion is stopped, the Gel will stay in place (thixotropic behaviour).

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- The primopattern LC Paste can be easily retained from its packing with a spatula or wax knife. Attention: the lid needs to be closed right away after having taken out the desired amount of material.

Processing and shaping are most efficiently done by hand.

The Paste is used where larger amounts of material need to be applied quickly (i.e. bars, pontics, etc.).

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· implant bar

First primopattern LC Gel is used to build up, light cure and finish the single copings over the implant posts. Attention: the screw channels need to be closed (wax).

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
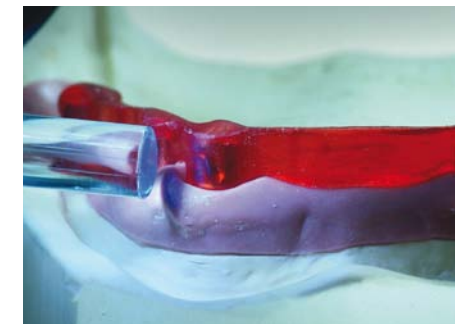
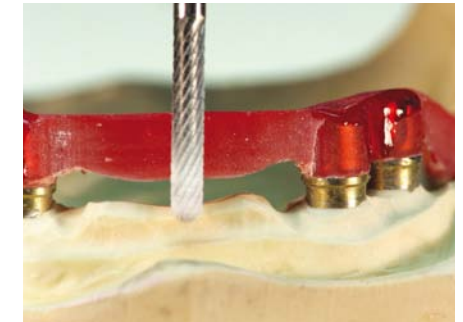
- The bar segments will be built up with primopattern LC Paste. To do so, an equivalent amount of Paste is pre-shaped into bar form with the fingers and placed between the copings. Then the complete modellation is light cured and afterwards the bar is milled.

Any corrections or re-enforcements will be done with the Gel. These corrections can be stabilized by pre-polymerising the relevant areas with a handheld lab light.

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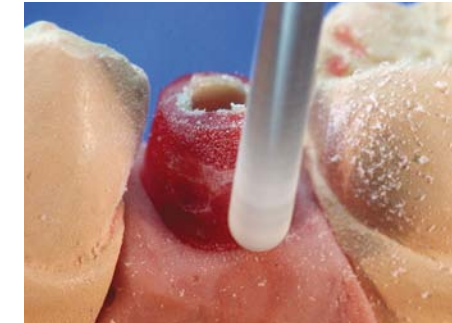
• **custom implant abutment**

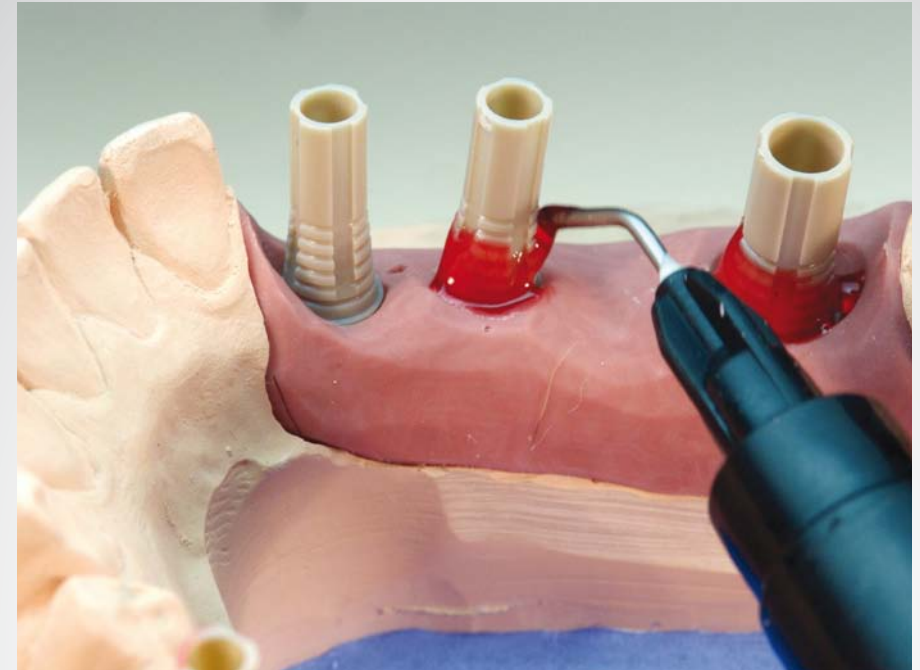In these cases primopattern LC Gel is used. It is no problem if the gel flows into the sulcus of the tissue model. It will not be polymerised in this area, as the light can't reach there. This way a clean marginal completion is reached automatically.

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• **implant-borne bridge restoration**

primopattern LC Gel is used for the abutments. For the pontic, primopattern LC Paste is the material of choice as it allows to much faster apply larger amounts at a time.

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- **transfer and insertion guide for custom zirconium implant abutments**

primopattern LC Paste is formed into rod shape and applied over the custom abutments. Of course, the screw channels need to be closed beforehand (wax). Since primopattern is extremely precise and dimensionally stable, the guide can be built up in one piece.

Relief cuts prior to or after polymerisation and/or reconnection after light curing are not necessary!

After light curing the transfer guide is lifted off the abutments and trimmed to shape. By trimming it will inevitably become thinner occlusally and labially. Consequently the guide should be adequately dimensioned lingually already during the modelling stage, in order to allow for the needed strength.

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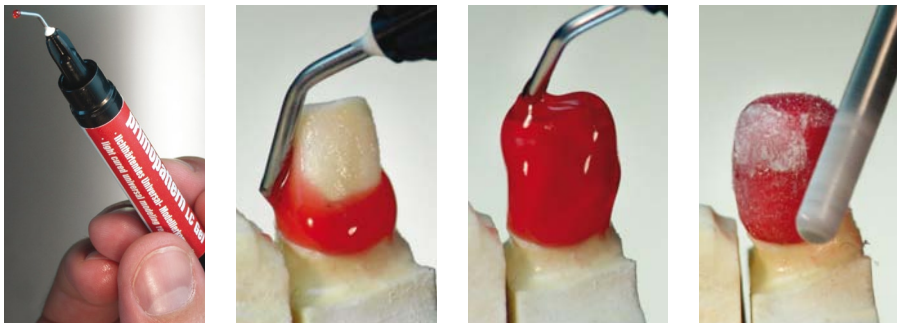
- **combined checking guide and bite check splint for six upper telescopic zirconium crowns**

In the same way as for the implant abutments transfer guide, again primopattern LC Paste is used in the case shown here. The manually pre-shaped rod is pushed onto the telescopic crowns and designed as needed. Next, the articulator is closed and the combined guide/splint is light cured, ideally in the articulator.

The perforation of this bite check splint in the canine area indicates that the actual bite situation of the patient needs to be attentively re-evaluated intraorally.

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• **crowns, copings or primary parts of any kind for casting, scanning, copy milling or pressing**

The stone dies will be isolated as described before. For modelling hold the syringe as shown in the picture and lightly press the piston. Depending on the case to be built up, an intermediate light curing followed by trimming and shaping could be done if needed.

When the syringe is used with a lightly vibrating motion, the material becomes more flowable, when the vibrating motion is stopped, the Gel will stay in place (thixotropic behaviour).

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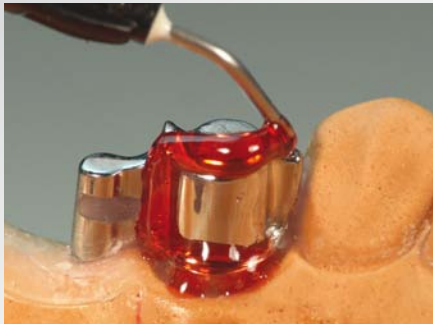
• **bridges of any kind for casting, scanning, copy milling or pressing**

First the copings over the adjacent dies are made with primopattern LC Gel. For the pontic, primopattern LC Paste is used.

Once the pontic is positioned properly, an intermediate polymerisation can be done, but is not obligatory. Further adjustments or refining is done with primopattern LC Gel. After final light curing, the primopattern understructure can be trimmed and shaped with carbide burs or rubber polishers as usual.



• **attachment techniques**

Secondary parts for attachment techniques are in general built-up with primopattern LC Gel. The metal or zirconium surfaces of the primary parts need to be smooth and clean. A separator is in most cases not necessary. In especially critical situations it can be useful to apply a thinnest layer of Vaseline on the primary surfaces. The secondary parts are lifted off after light curing with a dull instrument.

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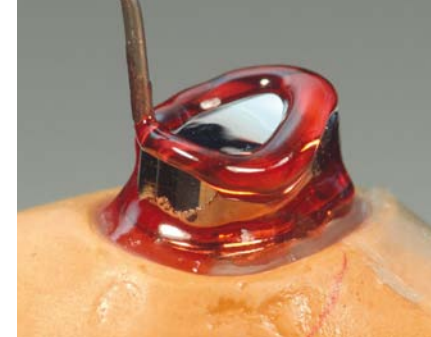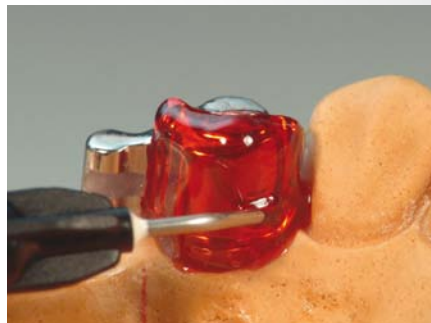• **secondary crowns for telescopic and conus crown techniques**

Secondary crowns are again built-up with primopattern LC Gel. The metal or zirconium surfaces of the primary crowns need to be smooth and clean. A separator is in most cases not necessary.

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Retention beads can be dispersed on the primopattern surfaces prior to light curing. The secondary crowns are lifted off after polymerisation with a dull instrument.

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general notice on primopattern LC Gel and primopattern LC Paste

- **composition:**
formulation of different acrylates and photo initiators
 - **delivery batch:**
primopattern LC Gel = 5 x 3g syringes,
15 hollow needles
primopattern LC Paste = 30g / pack
 - **working and storage notice:**
primopattern LC Gel and Paste should be used and stored at temperatures between 4 °C and 25 °C. The materials shall be protected against heat and direct sunlight. Always close the containers right after use (no sunlight or extensive laboratory light). The lot number and expiration dates for primopattern LC Gel and Paste are printed on the particular product labels. After the materials are expired they must not be used anymore.
 - **products warning:**
primopattern contains acrylates and initiators. In case any irritations occur by using the product, or if the user is aware that he might react to one of the products components, the material should not be used. Avoid direct eye or soft tissue contact with the unpolymerized material as well as with the oxygen inhibition layer, as this might, in rare cases, lead to sensitivities. If that should happen it is recommended to stop using the product.

For primopattern LC Paste in addition:
Classification Xi; R36/37/38. Irritating to eyes, respiratory system and skin.

To avoid any health risks, especially grinding and polishing should only be done at a work bench with an appropriate suction system
 - **Material Safety Data Sheets:**
The MSDS for primopattern LC Gel and Paste are deposited in the download area of www.primogroup.de. Alternatively they can be requested from the manufacturer (see back page) or the domestic distributor
 - **Regulations for use:**
All primopattern products were developed for the use in the dental field. They have to be used in accordance with this instruction manual. For any damages caused by the use of the products in any other than the dental field, neither the manufacturer nor the distributor will take any responsibility. In addition, the user is obliged to verify, at his own responsibility, whether the material can be used for the desired purpose, especially if the desired purpose is not described in the instruction manual.
 - **Caption:**

	lot number
	expiration date
	recommended storage temperature
	protect against light
- content, structure, text – Joachim Mosch
pictures – MDT Andreas Hoffmann, www.1dsz.de
design, layout – Christoph Weiß focusPOOL.com