



UNO•DUO™

Simplified Dual Cure Adhesive

DUO is the new dual cure catalyst for our award winning DenTASTIC UNO

Throw Away the Chemistry Set

Just mix 2 drops of UNO + 1 drop DUO for your all-purpose, dual cure adhesive.

DenTASTIC® UNO™

One Step Wet Bonding at a Price You'll Like
For all direct bonding light cure applications

DenTASTIC® UNO-DUO™

Simplified Dual Cure Adhesive
For indirect restorations and whenever self-cure or dual cure capability is indicated.

UNDO Kit: 6 mL UNO, 3 mL DUO, 5 mL Etch-Rite etching gel, 20 applicator tips, 50 brush tips, brush handle

UNO Kit: 2 x 6 mL UNO, 5 mL Etch-Rite etching gel, 20 applicator tips, 50 brush tips, brush handle


UNO-R 6 mL bottle DenTASTIC UNO

DUO DUO, 3 mL bottle

Manufactured under license. Patents developed by Rafael Bowen and assigned to the ADA Health Foundation

PULPDENT Corporation


80 Oakland Street • Watertown, MA 02471-0780 • U.S.A. • (617) 926-6666 or (800) 343-4342 • Fax (617) 926-6262
Visit Pulpdent Online at <http://www.pulpdent.com> • Email: pulpdent@pulpdent.com

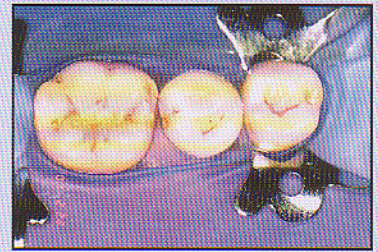


Preoperative



Apply Etch-Rite to the cavity prep for 15 seconds.

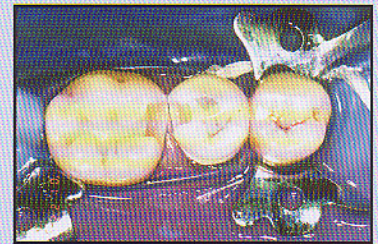

Rinse and leave dentin moist for wet bonding technique.

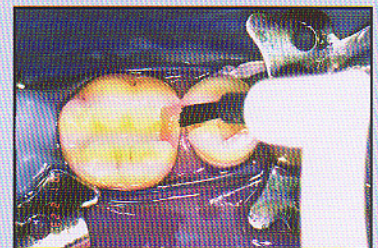

Use DenTASTIC UNO for light cure, or UNO + DUO for self-cure or dual cure applications.



Light cure DenTASTIC UNO for only 10 seconds.

Photos courtesy of
Dr. C.H. Pameijer

CE
0301