


NTI DIAMOND GRINDING AND FINISHING STRIPS

AS USED BY OLIVER BRIX

A routinely recurring situation. After firing, the contact points are too pronounced and a lot of patience and effort is required in preparing them to attain closed, surface area contacts.

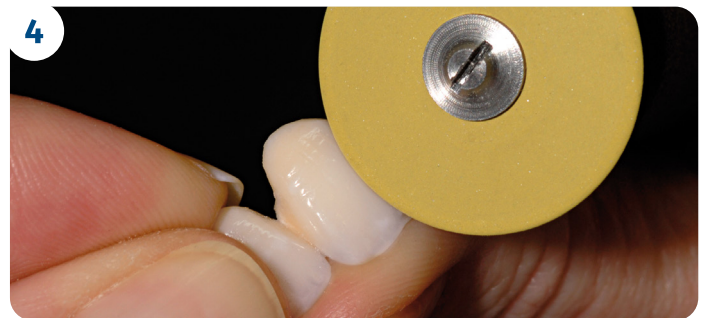

The 6mm wide diamond strip, coated on one side, precisely prepares a perfect contact point in minimum time. The diamond strip is placed between the crowns and moved back and forward several times until the high spot is reduced and an optimum contact point is attained.

There is a choice of three different grit sizes according to the situation and depending on the amount of early contact. Perfect interdental surfaces that any dentist or patient would be pleased with.

Final finish is attained by using the Yellow Ceraglaze (P30001) Re-glazing in the furnace is no longer necessary.

Motor speed 10,000rpm.

PACK OF 10 ASSORTED STRIPS

Order Ref: FS6-S0

Includes:

- 2 x FS6-M Medium Grit
- 4 x FS6-F Fine Grit
- 4 x FS6-SF Superfine Grit

£53.65

YELLOW CERAGLAZE DISC

Order Ref: P30001

£29.00



All prices exclude VAT and postage and are subject to change without prior notification.

Prestige Dental Ltd 7 Oxford Place, Bradford, West Yorkshire, BD3 0EF

Tel: 01274 721567

Email: info@prestige-dental.co.uk • www.prestige-dental.co.uk

If you prefer not to receive our special offers please contact us to unsubscribe

