

# Simplify the Mounting Process



Introducing  
**Whip Mix**  
**FlexiMount™**  
Universal Mounting  
Support


FlexiMount quickly and easily provides dynamic stability when mounting the lower casts.

You no longer need:

- Glue
- Paper Clips
- Accelerants

Its flexibility allows you to adjust as you go.

An additional advantage is you can attach models in less than 30 seconds to complete your mounting process.

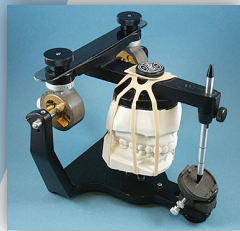

**Works with All Denar®,  
Hanau™ and Whip Mix®  
Articulators**

**Call your Whip Mix representative today  
for samples or additional information.**


**502-637-1451 800-626-5651**

**[www.whipmix.com](http://www.whipmix.com)**



RESTORATIVE ORAL HEALTH

# Dynamic Stability When Mounting Casts

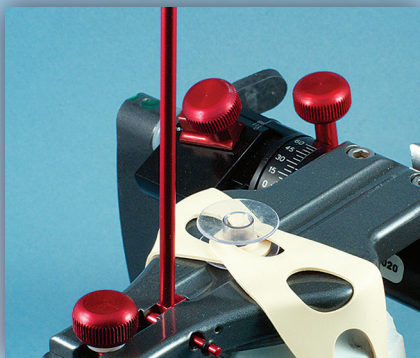

## FlexiMount Instructions

- Mount the upper cast as you would normally.
- Score the lower cast adequately and soak the base of the cast.
- Using the FlexiMount end with two holes, attach the FlexiMount device to the thumbscrew on the upper member.
- Open or invert the articulator so that you can relate the lower cast to the upper cast.
- If you are using an interocclusal record, seat it in place between the two casts.
- While holding the casts securely together, wrap the FlexiMount device around the casts and secure the other end to the thumbscrew on the upper member.
- Adjust the arms of the FlexiMount device if needed so that you have even pressure holding the casts together in good balance.
- Pull down to tighten individual arms of the device if needed for dynamic stability. This allows you to adjust the casts while keeping them in a stable position.
- Add mounting stone to the center of the cast and mounting plate.
- Once the stone has set, release the FlexiMount device from the upper member and remove the lower cast from the articulator.
- Using a sharp blade, cut the FlexiMount device away from the base of the cast.
- Add your final layer of mounting stone.

**NOTE:** For use with the Denar Mark 300 series articulators, use the clear button in your package in place of the thumbscrew on the upper member.

**For more information, or to view our step-by-step video, visit us at [whipmix.com](http://whipmix.com).**



Available in the following packages:

20011831 FlexiMount 3 pack

20011832 FlexiMount 20 pack

20011833 FlexiMount 100 pack



RESTORATIVE ORAL HEALTH

Whip Mix Corporation

361 Farmington Avenue, Louisville, KY USA 40209  
800-626-5651 • 502-637-1451 • Fax: 502-634-4512

[www.whipmix.com](http://www.whipmix.com)