


T-6 TABLE INSTALLATION AND ADJUSTMENT PROCEDURE

NOTE: For the following procedure, keep finger pressure on the table. Finger pressure should keep the table pressed onto the articulator.

1. With T-6 Table off the articulator, adjust the table wings to "0".
2. Identify the two adjusting screws and the bottom knob as shown below.

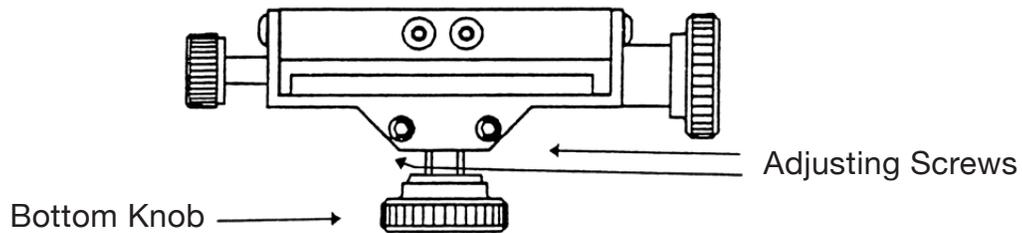

3. Adjust the two adjusting screws by backing them out (turn them CCW) until they are flush to approximately 1 mm out of the table.
4. Place table on articulator.
5. Set pin at "0".
6. Rotate table upward and forward to maximum (45°). Upper bows should not move upwards visibly; but it will in most cases.
7. If there is movement of the upper bow, begin adjusting the set screws turning them in, (CC - Clockwise) each in equal amounts. (i.e. half a turn at a time)
8. After each adjustment, check for upper bow movement by rotating the table.
9. If upper bow movement is observed, repeat steps 7 and 8.
10. When adjustment is reached (no upper bow movement when table is rotated), tighten the bottom knob and release finger pressure.



RESTORATIVE ORAL HEALTH

Whip Mix® is a registered trademark of Whip Mix Corporation.