Ultra-Waxer²

34844

Directions for Use

**Kerr**
Sybron Dental Specialties

ENGLISH

Thank you for purchasing the new Ultra-Waxer 2, the most advanced electronic waxing pen of its kind. The Ultra-Waxer has been the industry standard for dental laboratory applications for decades.

Features

- Comfortable color-coded cushion grip tips
- Four programmable temperature presets
- Quick Heating of tips
- Dual pens for simultaneous use
- Adjustable Auto Off
- Dual Voltage
- Temperature Range: 97°F to 500°F (36°C to 260°C)

1. Specifications

Height: 3.25" (8.26 cm)

Width: 7.5" (19.05 cm)

Depth: 7" (17.78 cm)

Weight: 1.6 lb (0.726 kg)

Electrical Rating:

100/240 Volts, 1.0 Amps,
30 Watts, 50/60 Hertz

Ambient Temperature:

41°F (+5°C) to 104°F (+40°C)

Relative Humidity: 80% Max

Altitude Limit: 2000 meters

Installation Category II

Pollution Degree 2

Indoor Use Only

Mains supply voltage fluctuations not to exceed 10 percent of the nominal supply voltage

2. Package Contents

Power Unit (1)

Ultra-Tips #35167 (1), 35168 (2)

Ultra-Spatulas #35177 (2)

Power Cord US Plug #34849 (1)

Power Cord EU Plug #34850 (1)

Instruction Manual (1)

3. Installation

Place unit on a stable work surface with appropriate ventilation approximately 6" (15.24 cm) from walls or other equipment. Confirm that electrical ratings listed on the specifications label meet your electrical needs.

Connections

Power Connection: The Ultra-Waxer 2 is a dual voltage unit. Simply plug the correct IEC 320 Country specific power cord into the back of the unit.

Spatula Connection: Connect the two Ultra-Spatulas, by inserting the Spatula plugs into the two Spatula jacks on the front of the unit.

4. Touch Pad Operation

Touch Pad contains 4 touch buttons for quick and easy navigation of the features.

ENTER button for power on/off, and for selection
UP/DOWN button to scroll through options
MENU button to access many applications

Menu Options

Quick Heat - supplies quick heat to a cold tip

Preset Temp - ability to set 4 programmable temperatures

Select C or F – choose between Celsius and Fahrenheit

Auto Off – Enable/Disable and adjust Auto Off time

Buzzer – Enable / Disable buzzer sound

Turn Off UW – Turn off Ultra-Waxer

(See detailed Touch Pad instructions for more information).

Default Settings

Preset Temperatures

Preset Temp 1 93°C/200°F

Preset Temp 2 149°C/300°F

Preset Temp 3 204°C/400°F

Preset Temp 4 260°C/500°F

Spatula Temperatures 97°F

Temperature unit °F

Auto Off Time 90 minutes


Buzzer Enabled

Reset Button

To return to the Default Settings, press the Reset Button located on the left side of the unit.

5. Spatula / Tip Operation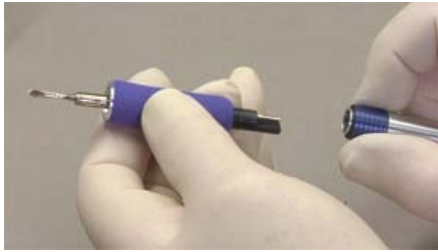

Removing a Tip

Retract the spring loaded Quick Connect Collar (Blue) towards the cord end of the Ultra-Spatula handle. The movement of the collar will be about 0.3 cm. Gently remove the Heating Tip in the opposite direction.

CAUTION Handle a hot tip with care to prevent burns.

Replacing a Tip

Retract the Quick Connect Collar (Blue) and insert the Heating Tip about half way. Gently rotate the Heating Tip until you locate the connection (two flat sides) that will allow the Heating Tip to be inserted the rest of the way. Use the Quick Heat feature (under Menu) once you have changed to a new tip, the Quick Heat feature will bring the cold Tip to a working temperature quickly.

Ultra-Waxer Heating Tips


1 2 3 4 5 6 7 8 9 10

1. 35167 Small PKT, Blue
2. 35168 Large PKT, Teal
3. 35169 Beavertail, Purple
4. 35170 Small #7, Grey
5. 35171 Large #7, Green
6. 35172 Needle, Red
7. 35173 ½ Hollenback, Black
8. 35174 Small Denture, Pink
9. 35175 Large Denture, Orange
10. 35176 Denture Spoon, Yellow
- 35177 Ultra-Spatula Cord/Handle

6. Application

Crown & Bridge Applications

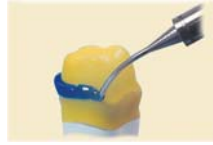Inlays, onlays, full crowns, ceramic frameworks.

Fill undercuts, cover sharp edges.

Assemble multiple unit bridges, attach sprues.

Seal margins of copings, crowns, inlays and onlays.

CAUTION

1. Read complete Directions for Use before operation.
2. Confirm unit is connected to a grounded power source.
3. Keep unit away from water and moisture.
4. Do not touch heated tips when in use.
5. Intended for use with traditional waxes only.
6. Avoid allowing the heated tips to touch spatula wire.
7. Do not block the ventilation opening in rear of unit.
8. Keep heated tips in holder to avoid damage or injury.
9. Do not attempt to repair or disassemble the unit.
10. Contact Kerr Corp. for repair and/or troubleshooting.

HOT SURFACES - Do not touch external and internal exposed parts of the unit.

ALTERNATING CURRENT - An electric current that reverses its direction at regularly recurring intervals.

LIMITED WARRANTY

LIMITATION OF KERR'S LIABILITY

Kerr's technical advice, whether verbal or in writing, is designed to assist dentists and laboratories in using Kerr's product. The dentist or laboratory assumes all risk and liability for damages arising out of the improper use of Kerr's product. In the event of a defect in material or workmanship, Kerr's liability is limited, at Kerr's option, to replacement of the defective product or part thereof, or reimbursement of the actual cost of the defective product. In order to take advantage of this limited warranty, the defective product must be returned to Kerr. In no event shall Kerr be liable for any indirect, incidental, or consequential damages.

EXCEPT AS EXPRESSLY PROVIDED ABOVE, THERE ARE NO WARRANTIES, BY KERR, EXPRESS OR IMPLIED, INCLUDING WARRANTIES WITH RESPECT TO DESCRIPTION, QUALITY, OR FITNESS FOR A PARTICULAR PURPOSE.

Action	Button Selection / Screen View	Info
Power On	 KERR ULTRA-WAXER	Press and release ENTER button to power on unit.
Select Spatula	 SPATULA1 97F SPATULA2 97F < 	Press and release MENU button to select spatula. Press ENTER to select
Adjust Temperature	  SPATULA1 97F SPATULA2 150F < 	Press UP/DOWN buttons to change temperature, and press ENTER to select.
Menu Options	 QUICK HEAT PRESET TEMP <	Press MENU button for 2 seconds to get to menu options.
Quick Heat	 QUICK HEAT PRESET TEMP <  SPATULA1 97F < SPATULA2 97F   Quick Heats Spatula 1   Quick Heats Spatula 2	Press ENTER to select Quick Heat, spatula selected will blink for 15 seconds until heated. Also access Quick heat quickly by pressing MENU & UP, or MENU & DOWN buttons simultaneously.
Preset Temperature	 PRESET TEMP < SELECT C OR F   S1 PRESET TEMP1 <  S1 PRESET TEMP2   S1 PRESET TEMP1 < 200F    SPATULA1 200F < SPATULA2 300F 	To select Preset temperatures, select the spatula. Adjust to desired temperature by using UP/DOWN buttons, then press ENTER to select.
Select °C / °F	 SELECT C OR F < AUTO OFF   DEG CELSIUS DEG FAHRENHEIT < 	To change from °C to °F, use UP/DOWN buttons then press ENTER to select.
Auto Off	 AUTO OFF BUZZER <   DISABLE EDIT AUTO OFF <    AUTO OFF ENABLE 90 mins    AUTO OFF ENABLE 	Enable or Disable the Auto Off feature by using ENTER button. Edit Audit Off cutoff time by using UP/DOWN buttons to change minutes, then select ENTER.
Buzzer	 BUZZER TURN OFF UW <   BUZZER ON BUZZER OFF 	Turn on or off Buzzer in menu options, by selecting and pressing ENTER button.
Power Off	 BUZZER TURN OFF UW <  TURNING OFF ULTRA-WAXER	Select TURN OFF UW from menu, or Press ENTER button for 3 seconds to power off.

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Kerr Italia S.r.l.
Kerr Australia Pty. Ltd.
Sybron Dental Specialties Japan, Inc.